



- Compatible with Reach at-home alarms, Advent xt warden call, Altec Response and Touchsafe Pro Nursecall systems
- Large momentary action pushbutton
- Ultra low profile (10mm at lowest point)
- Easily operated across whole area
- Actuation force: 180 grams nominal
- Splash proof – do not use in damp/humid conditions
- Low battery warning
- Dimensions: 125mm diameter x 26mm high
- Weight: 110 grams
- Product Code: ZXT439

### **Where should Big Buttons be located?**

The big button is a free-standing device which can be placed on a table within reach of the user. One press of the button will initiate an emergency alarm call to a local Carer or remote control centre. This product is suitable for persons with mobility or dexterity difficulties.

### **Installation**

Select the best location, and then simply place the big button on the table, cabinet, shelf, etc.

### **Registering the Big Button**

It is necessary to register a new big button onto the warden call system, at-home alarm unit or Nursecall system. See the following pages for how to put each of these products into registration mode.



*The range of all radio devices can be affected by the working environment - always take care during installation and perform several test calls from all extremes of the building.*

### **Battery Life**

The big button battery is checked once a day, if the voltage falls and stays below a preset level for 7 consecutive days a low battery alert is automatically transmitted. Once a low battery call is received the big button should be replaced within 7 days. Big buttons must be returned to Tynetec/Aidcall for battery replacement.

Detailed information on battery management for all Tynetec/Aidcall products is available - request Doc No. FM0630.


### **Maintenance**

The big button should be activated once a month to check the battery and verify operation.

The unit can be wiped clean with a damp cloth, never use detergents or polish and do not fully submerge in water.

### Registering a Big Button onto a Reach at-home alarm unit

1. Put the Reach unit into radio device learn mode...

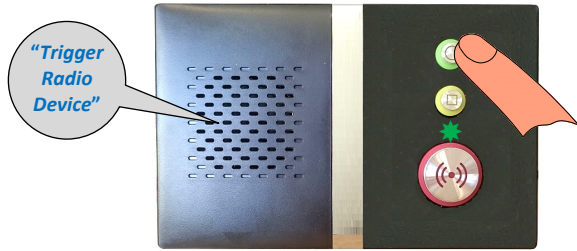

Press and HOLD the **GREEN** button on the Reach unit.

When the front light **FLASHES FAST GREEN** release the button and the Reach will announce "**Trigger Radio Device**".


You now have 1 minute to activate the big button.

2. Activate the big button...



Press the big button once and the Reach will announce "**Pendant**".

3. Exit radio device learn mode...

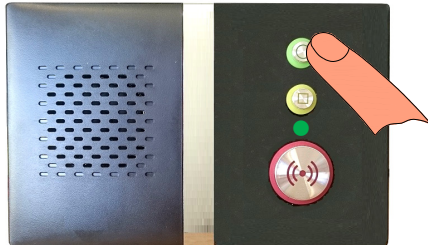

Press the **GREEN** button on the Reach unit.

The front light will return to **STEADY GREEN** and the Reach is back in normal operating mode.

4. Check with the Control Centre that it is convenient to perform a test call, press the big button and confirm they receive a pendant alarm call.

### Deleting a Big Button from a Reach at-home alarm unit

Enter **\* 1 6 7 0** using the keypad on the rear of the Reach unit – the front light will **FLASH FAST GREEN** and the unit will announce "**Program Mode - Ready**".

Enter **# 0 4 1** and the Reach will announce "**Delete Radio Device**" followed by a "**Beep**".

Enter the 8 digit big button ID to be deleted; after the "**Beep**" press **#** to exit.

Press the **\*** key to exit Program Mode; the unit will sound a "**Double-Beep**" and the front light will return to **STEADY GREEN**.

<sup>‡</sup> The 8 digit ID can be found on a label on the back of the big button – it will always start with 20xxxxxx



The instructions given above for the standard Reach product also applies to the Reach Plus, Reach Plus GSM and Reach Ultra models. Refer to the instruction booklets provided with these products for more information on programming and testing radio devices.

### Registering a Big Button onto an Advent xt warden call system

1. Put the Advent xt system into radio device learn mode...



Using the Manager's DECT telephone;  
Press the **LINE** key and wait for the "**Ready**" prompt.  
Enter **5 0 0 0** followed by the **\*** key.  
Enter the flat number the big button is being installed in followed by the **\*** key.  
You now have 1 minute to activate the big button.

2. Activate the big button...



Press the big button once.  
Wait for the "**Enter Location Code**" prompt then press **0** (no location) followed by the **\*** key.  
Wait for the "**Pendant Enabled – Enter Flat**" prompt.

3. Exit the Advent xt system radio device learn mode...



Using the Manager's DECT telephone;  
Press the **#** key and wait for the "**Ready**" prompt.  
Press the **LINE** key to clear the connection and the warden call system is now back in normal operating mode.

4. Activate the big button and check a pendant alarm call is reported on the warden call system.

### Deleting a Big Button from an Advent xt warden call system

Press the **LINE** key and wait for the "**Ready**" prompt.  
Enter **5 0 0 1** followed by the **\*** key.  
Enter the flat number the big button is being deleted from followed by the **\*** key.  
Activate the big button to be deleted.  
Wait for the "**Pendant Disabled – Enter Flat**" prompt.  
Press the **#** key and wait for the "**Ready**" prompt, then press the **LINE** key to clear the connection.



Refer to the Advent xt operators manual (Doc No. FM0420) provided with the warden call system for more information on programming and testing radio devices.

### Registering a Big Button onto an Altec Response local carer alarm unit

1. Put the Altec Response unit into radio device learn mode...



Press **ENTER** to activate the wake-up display then **ENTER** again to display the main menu.

Press **ENTER** to select **→ Radio Devices ←**

Press **ENTER** to select **→ Learn Device ←**

The display will prompt you to **“Trigger the Sensor”**.

You now have 20 seconds to activate the big button.

2. Activate the big button...



Press the big button once.

The display will show **Personal Pendant** and the big button 8 digit ID number.

Check the ID is correct, press the **▼** key to select **→ Accept ←** then press **ENTER**.

**Note: if another device has been picked-up press ENTER to → Reject ← and try again.**

The display will show **← Flat →**, use the **◀ & ▶** keys to choose the dwelling type.

Press the **▼** key to select **→ No: 000000 ←** then press the **ENTER** key.

Use the **▲ & ▼** keys to select each digit of the flat number and the **◀ & ▶** keys to move between digits then press the **ENTER** key.

Press **▼** key several times to select **→ Exit ←** then press the **ENTER** key.

The display will prompt you to **“Trigger the Sensor”** again.

3. Exit the Altec Response radio device learn mode...



From the **Learn Device** menu select **→ Exit ←** then press **ENTER**.

From the **Radio Devices** menu select **→ Exit ←** then press **ENTER**.

The display will return to the wake-up screen and go blank after 10 seconds.

The Altec Response is now back in normal operating mode.

4. Activate the big button and check a pendant alarm is displayed on the Altec Response unit.

### Deleting a Big Button from an Altec Response local carer alarm unit

Press **ENTER** to activate the wake-up display then **ENTER** again to display the main menu

Press **ENTER** to select **→ Radio Devices ←**

Press the **▼** key to select **→ Delete Device ←** then press **ENTER**

The display will prompt you to **“Trigger the Sensor”**

Activate the big button to be deleted - the display will show the device type and its 8 digit ID number

If this is the device you activated press the **▼** key to select **→ Delete ←** then press **ENTER**

If it's NOT the device you activated press the **ENTER** key to select **→ Reject ←** then try again


To exit the mode select **→ Exit ←** then press the **ENTER** key.




Refer to the Altec Response installation guide (Doc No. FM0548) provided with the portable alarm unit for more information on programming and testing radio devices.

## Registering a Big Button onto a Touchsafe Pro Nursecall system

1. Put the Touchsafe Pro Master Panel into Telecare registration mode; touch the screen and Logon as an Engineer or a Manager (password required), then from the main menu page touch the **Telecare** button to open the screen shown below...

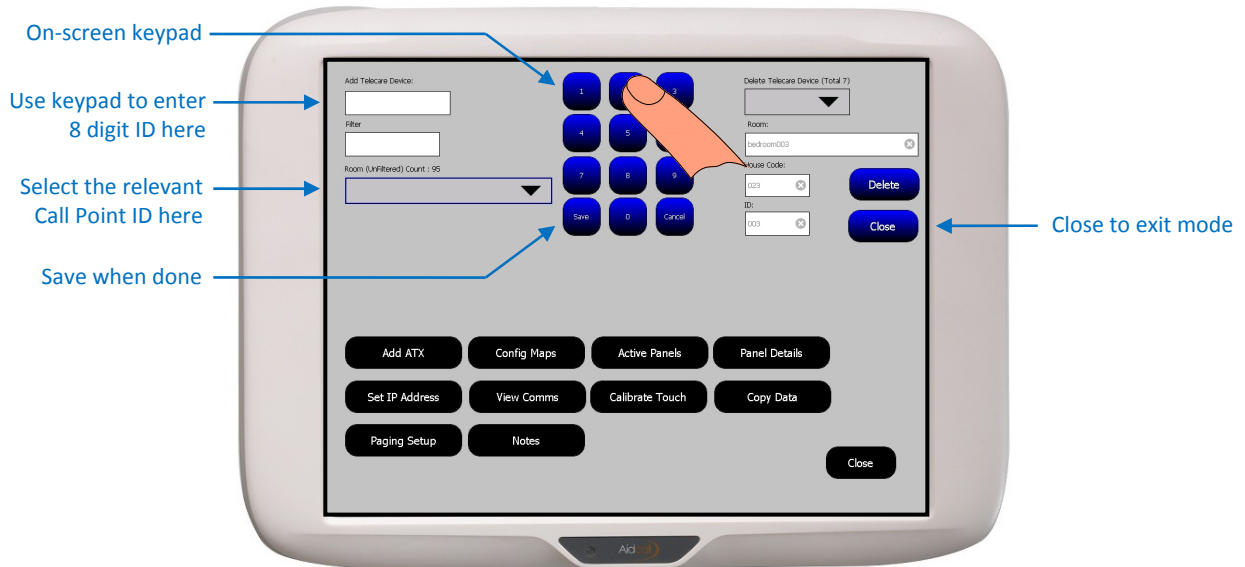

2. Enter the big button ID...

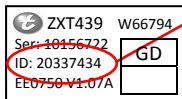

The 8 digit ID is printed on a label on the underside of the big button.  
*Note: this ID will always start with 20xxxxxx.*

Touch the **Add Telecare Device** box then use the on-screen keypad to enter the ID.

*Hint: enter a zone in the **Filter** box to narrow the search for the room (see below).*

Select the users Call Point ID from the **Room** pull-down list box then press **Save**.

*Note: this is the room unit that will be used to cancel the call.*

3. Press the **Close** button to exit Telecare mode and the Nursecall panel is now back in normal operating mode.

4. Activate the big button and check a pendant call is displayed on the Nursecall panel like this...



The screen banner is **orange** for a pendant call (Touch the banner to display notes).

Press the **Accept** button on the banner to silence the alarm - this will also notify all other panels the call is being dealt with.

The screen banner changes to **purple** when a call is accepted.

You must visit the appropriate Nursecall Call Point to cancel the call.



Press the **LIGHT GREY** button to cancel the call

## Deleting a Big Button from a Touchsafe Pro Nursecall system

Select the big button ID to be deleted from the **Delete Telecare Device** pull-down list box (prefixed with a "T20").

Details of the big button selected will be displayed in the **Room**, **House Code** and **ID** boxes below.

If the details shown are correct press the **Delete** button.

Press the **Close** button to exit.



Refer to the Touchsafe Pro Display Panel user guide (Doc No. FM5505) provided with the Nursecall system for more information.

Tynetec and Aidcall operate a policy of continuous improvement and reserve the right to change product specifications without notice. If any variation to the details contained in this document are suspected please contact Tynetec's customer support dept on 01670 352371.